

SMART LIBRARIES




INSTRUCTIONS

1. DOWNLOAD SMART LIBRARY FOR 3Shape


2. IMPORT SMART LIBRARY FOR 3Shape

1.DOWNLOAD SMART LIBRARY FOR 3Shape

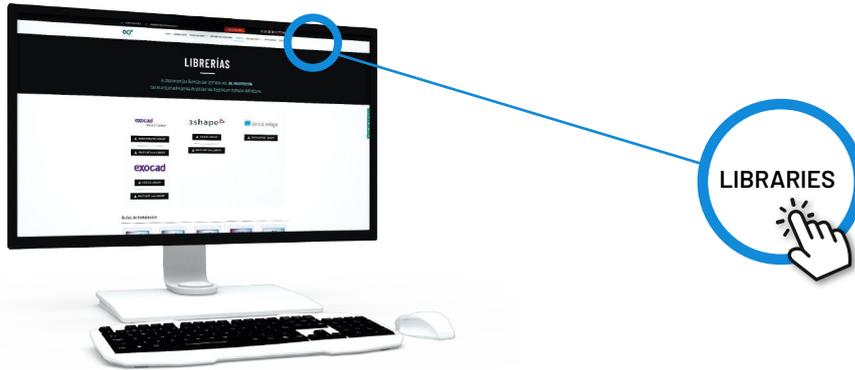


STEP 1

1. From a computer, connect to: www.smartimplantsolutions.com
2. Click the **LIBRARIES** button in the main menu.

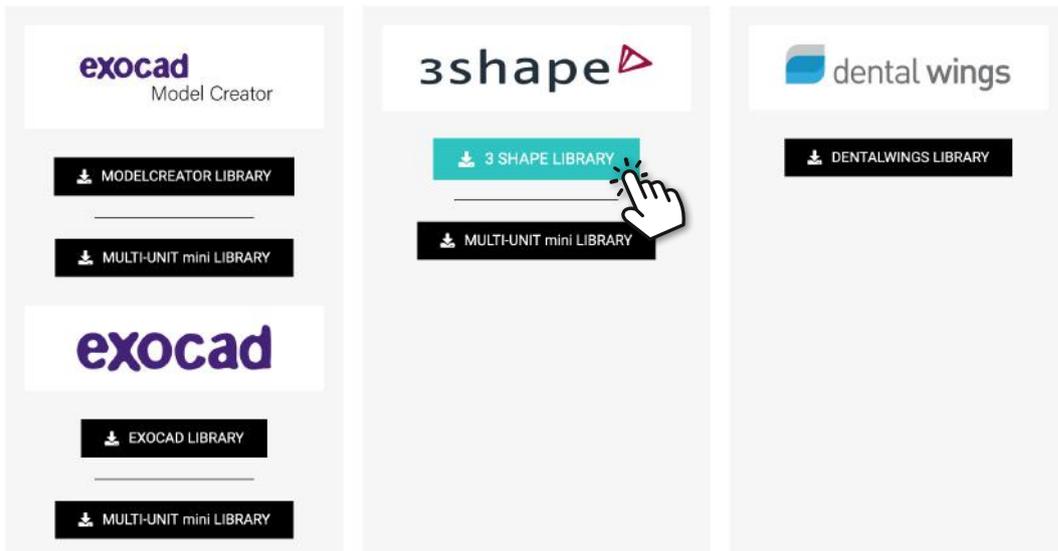

STEP 2

1. Click on the library button to download.
This link takes you directly to the **DROPBOX** download page.

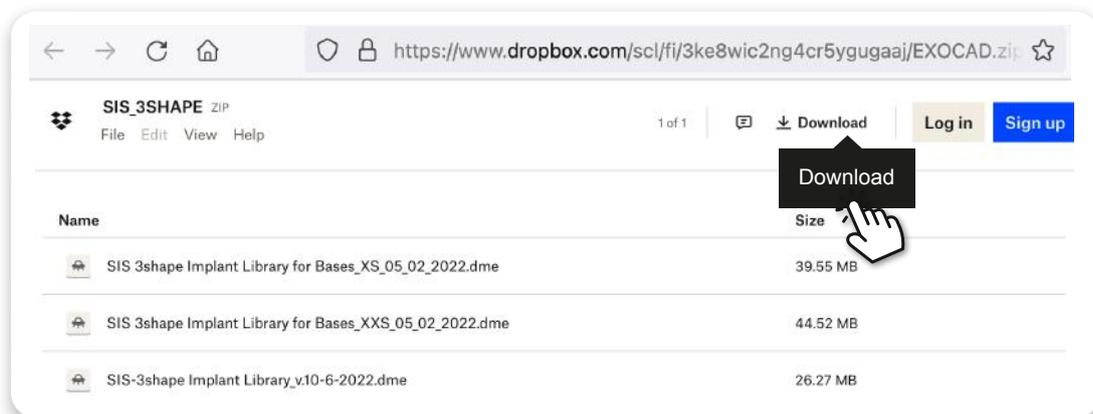

STEP 3

DROPBOX:

1. Press the **DOWNLOAD** button
The file will be downloaded to the Downloads folder on your PC.

2.IMPORT SMART LIBRARY FOR 3Shape

UNZIP THE FILE .ZIP

1. Go to the Downloads folder on your PC.
2. Unzip the downloaded file.




To unzip the file you can use a standard zip file decompression program downloaded from the internet such as WinZip or WinRAR.


Open Control Panel



You will see the following window.

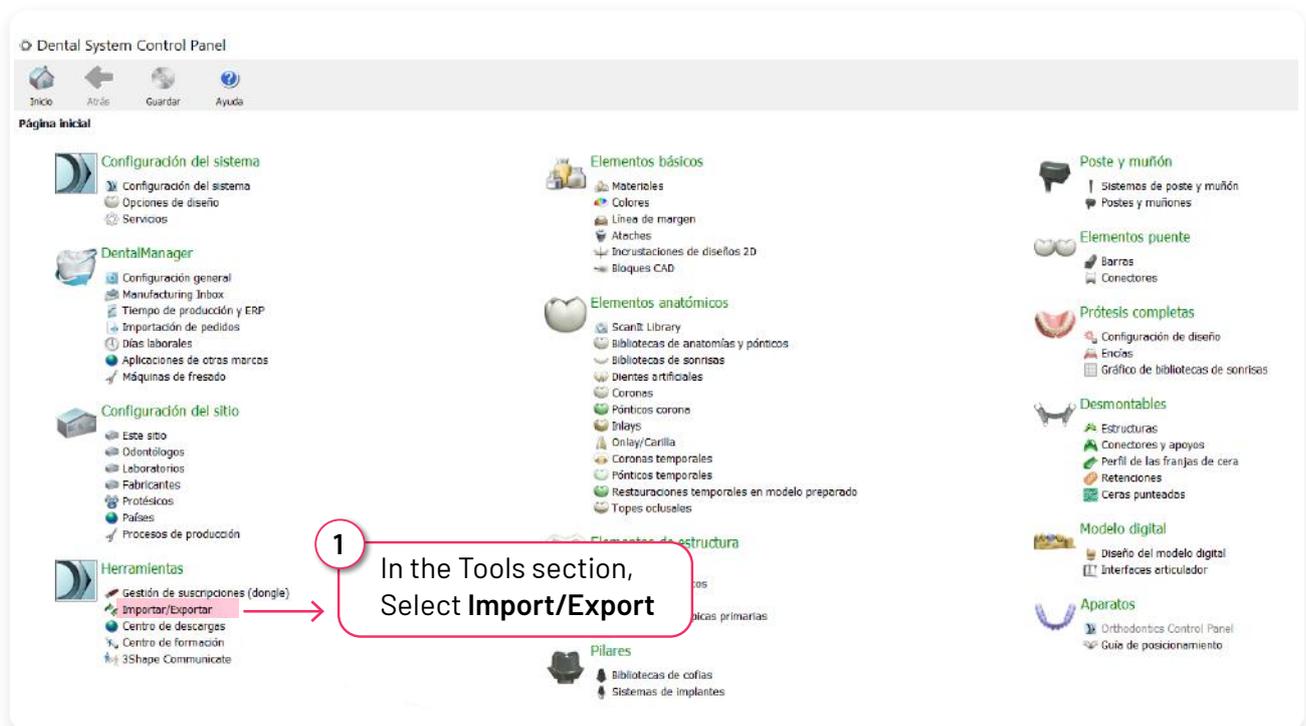


The following window is opened.

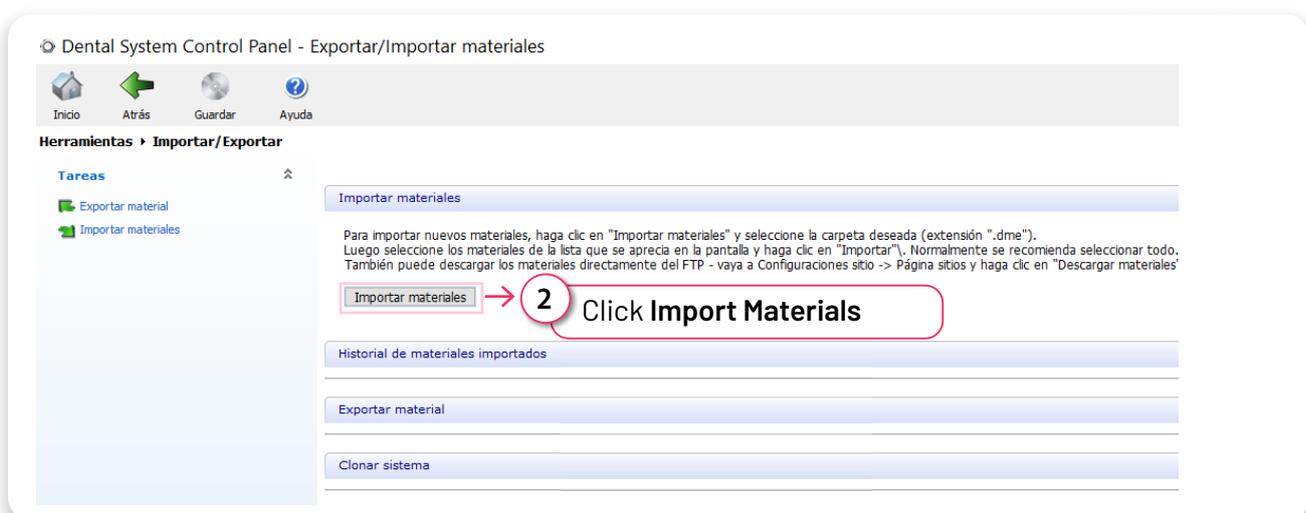


2.IMPORT SMART LIBRARY FOR 3Shape

A new window is opened.

In this window **find the path** where you have the **downloaded file and select it**.

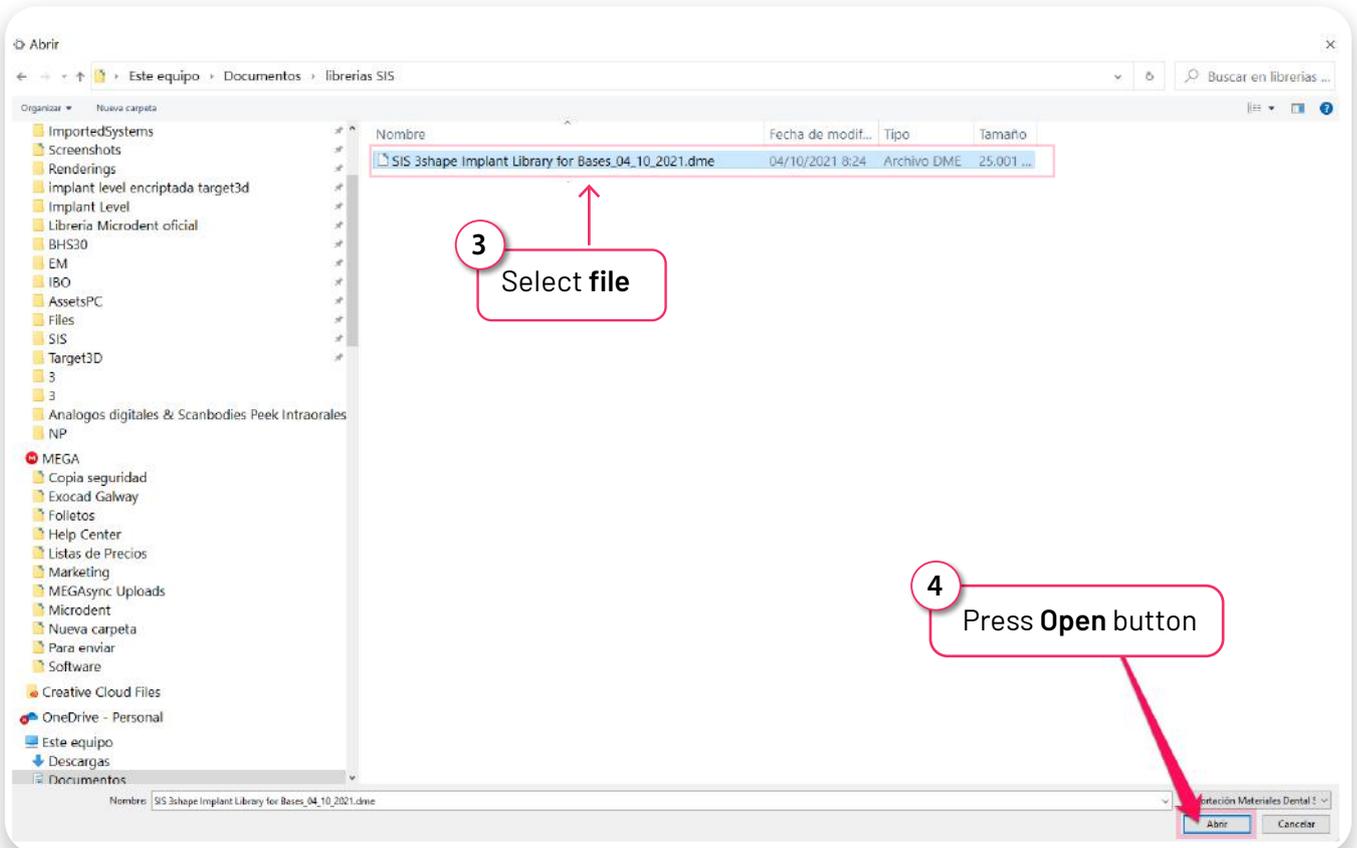

A popup window is opened.

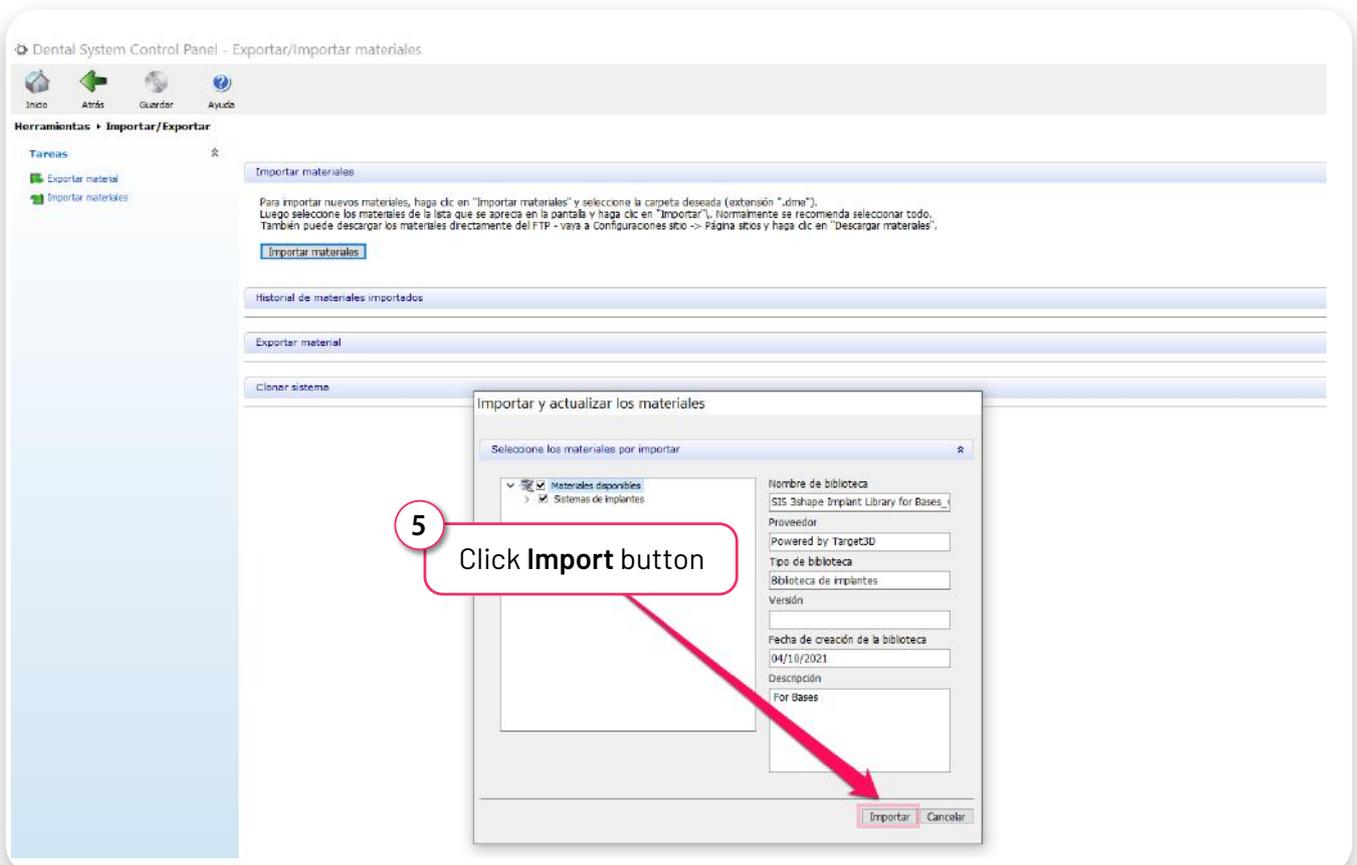

2.IMPORT SMART LIBRARY FOR 3Shape

The libraries are installed and ready to use.

IMPORTANT

Finally you must **close the software** (restart 3Shape software) for the changes to take effect.

