

# SMART LIBRARIES




## INSTRUCTIONS

1.DOWNLOAD MODELCREATOR LIBRARIES

---


2.INSTALL MODELCREATOR LIBRARIES

# 1.DOWNLOAD MODELCREATOR LIBRARIES

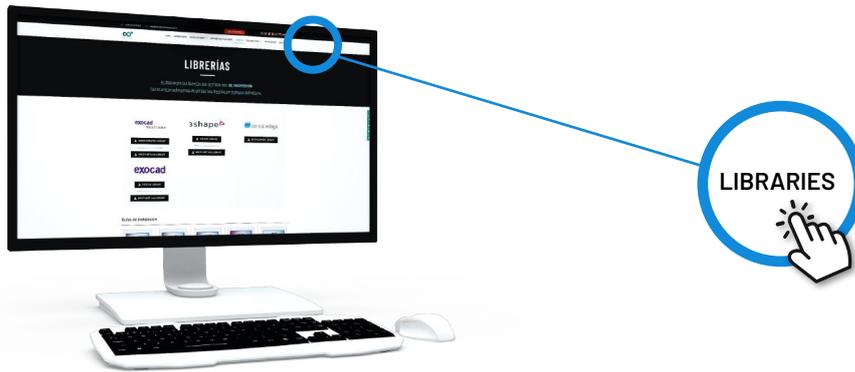


## STEP 1

1. From a computer, connect to: [www.smartimplantsolutions.com](http://www.smartimplantsolutions.com)
2. Click the **LIBRARIES** button in the main menu.

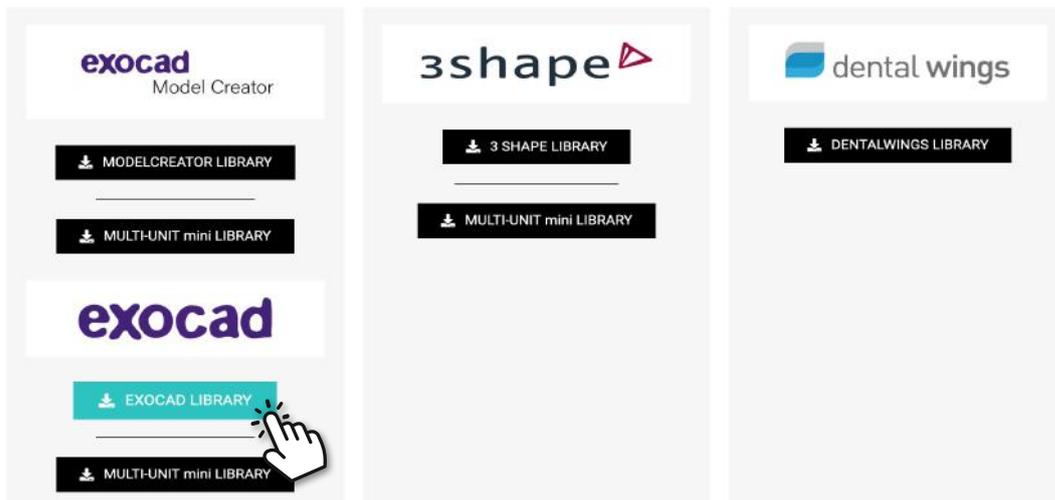

## STEP 2

1. Click on the library button to download.  
This link takes you directly to the **DROPBOX** download page.

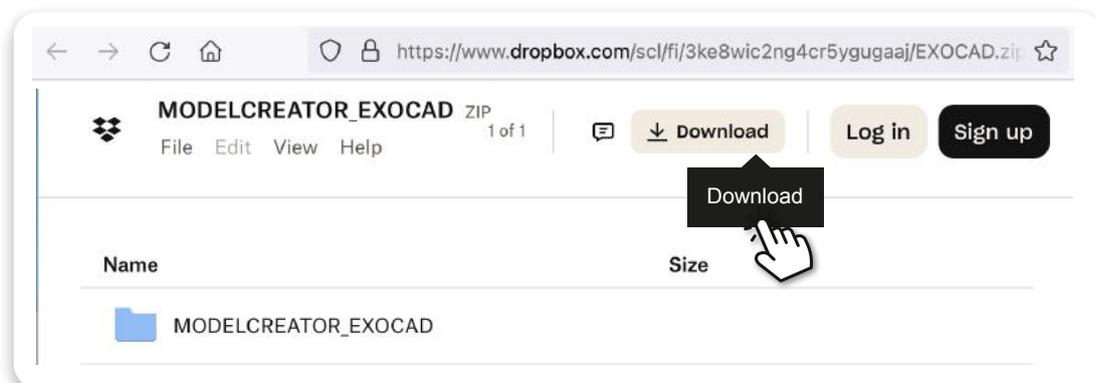

## STEP 3

### DROPBOX:


1. Press the **DOWNLOAD** button  
The file will be downloaded to the Downloads folder on your PC.

## 2.INSTALL MODELCREATOR LIBRARIES

### STEP 1

#### UNZIP THE FILE .ZIP


1. Go to the Downloads folder on your PC.
2. Unzip the downloaded file.

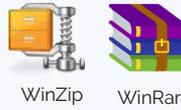

To unzip the file you can use a standard zip file decompression program downloaded from the internet such as WinZip or WinRAR.

### STEP 2



When you unzip the file you will get a folder called "SIS\_MODELCREATOR\_LIBRARY..." that contains:

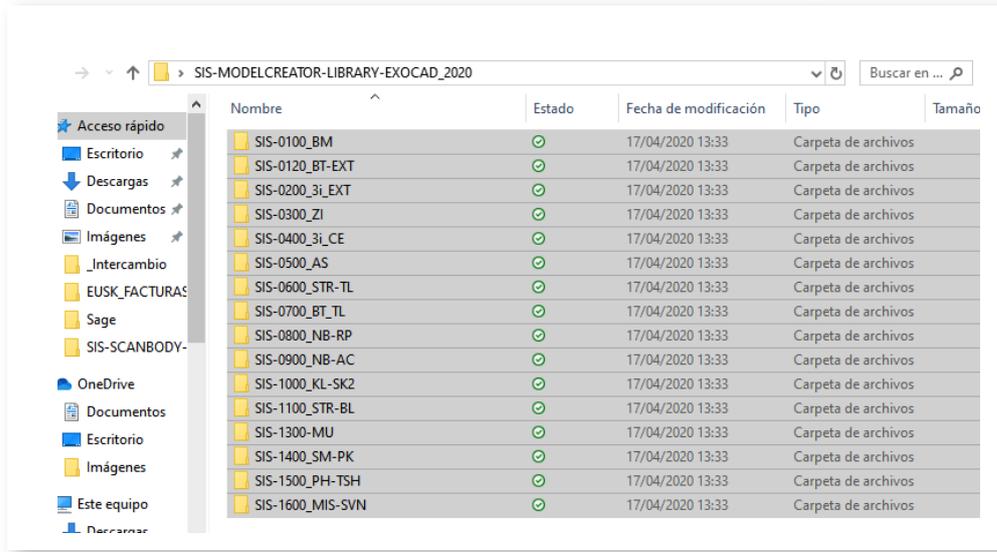

### STEP 3

#### COPY FOLDERS

1. Open the folder "SIS\_MODELCREATOR\_LIBRARY-EXOCAD...."
2. Select and copy all folders.
3. Paste them into the following path **C:/exocad/DentalCADApp/library/modelcreator/implants**

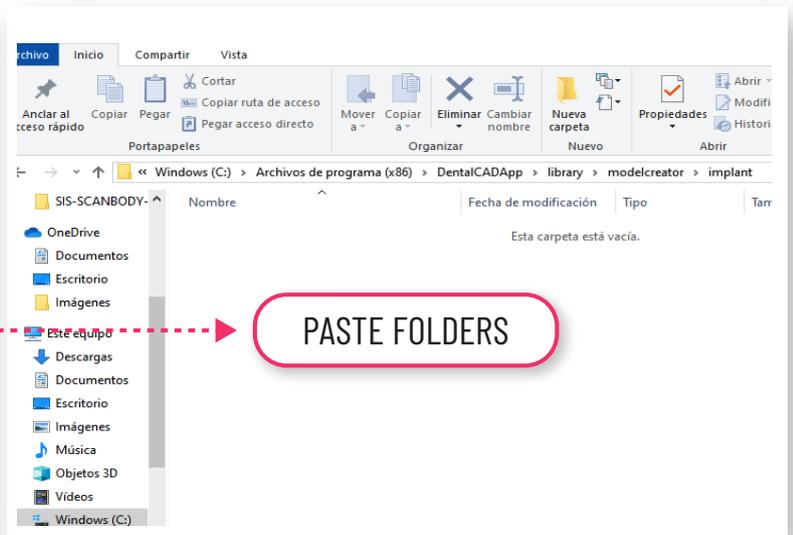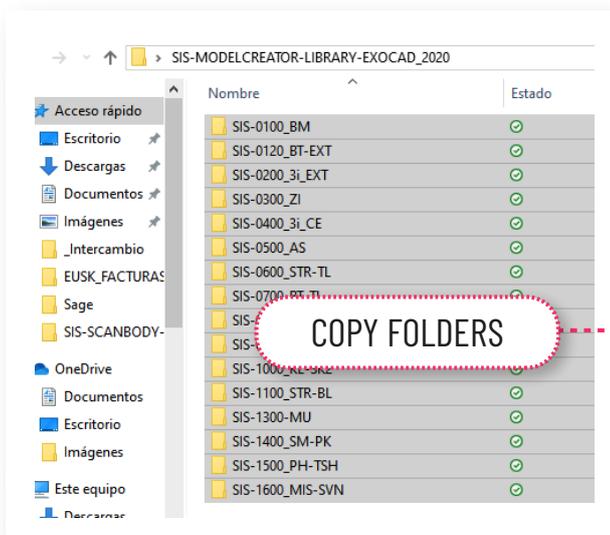

**SHORTCUT:** Using file manager of your computer, you can find required path very quickly.

**C:/exocad/DentalCADApp/library/modelcreator/implants**

## 2.INSTALL MODELCREATOR LIBRARIES

---

### STEP4

#### CHECK

Make sure you have the libraries correctly copied in the Exocad library menu.  
Go into **implants** folder and be sure that files are copied

### STEP 5

#### EXECUTE EXOCAD

